

Footrest Extension Instructions



Scan with phone for video installation instructions.

What's included?
(Allen wrench found in your tool kit)



Step 1: Separate the longer screws from the shorter screws.

Step 2: Situate the plate so that the threads holes are on the bottom (notice that there are no threads on top).

No thread--

Threads--



Step 3: Place the long screws through the top of the plate (non-threaded portion), slide the spacers onto the other side, then tighten.

Note: Ensure that the plate is straight or you may strip the threads.



Step 4: Insert the short screws through the holes on the footrests and attach to the bottom portion of the extension plate.



Final result:

For questions, please call us at:

800-342-2214

